


Rectal/Perirectal Specimen Collection


1

Use Cepheid Collection Device #900-0370 to collect the specimen.

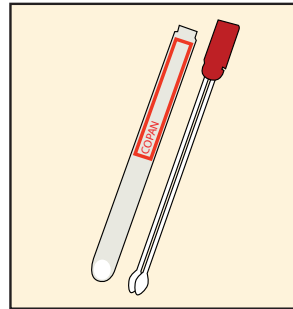

3


Place the swab pair back into the transport tube.

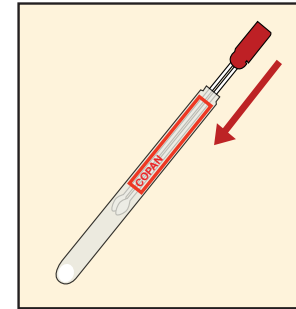

2

Rectal Swab Sample


Carefully insert both swab tips approximately 1 cm beyond the anal sphincter and rotate gently.

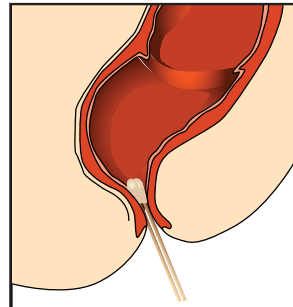

OR

Perirectal Swab Sample

Carefully insert both swab tips no more than 1 cm into the anal opening before the anal sphincter and rotate gently.

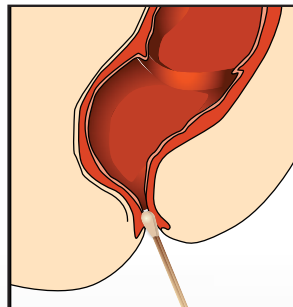

4

Swabs in the transport tube can be stored at 15-28 °C for up to five days.

