

# Xpert<sup>®</sup> Factor II & Factor V Cartridge Preparation



Refer to the package insert for detailed instructions, precautions, and warnings.

For a copy of the SDS, visit [www.cepheid.com](http://www.cepheid.com) or [www.cepheidinternational.com](http://www.cepheidinternational.com)

Cepheid Technical Support  
US office  
(888) 838-3222, Option 2  
[techsupport@cepheid.com](mailto:techsupport@cepheid.com)

European office  
+33 563 82 53 19  
[support@cepheideurope.com](mailto:support@cepheideurope.com)

1 Obtain the EDTA or sodium citrate sample collection tube.




2 Obtain one Xpert Factor II & Factor V cartridge.




3 Gently mix sample by inverting the sample tube 5 times, until homogeneous.



4 Open the Xpert cartridge lid.

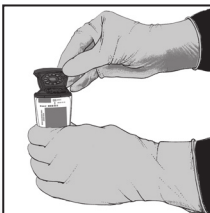

5 Using a volumetric pipette with an aerosol resistant tip, aspirate 50ul. of the sample.

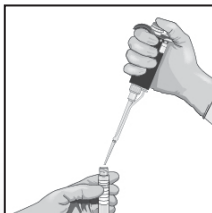

6 Transfer the sample to the bottom wall of the "S" chamber of the cartridge.



7 Close the Xpert cartridge lid.



8 Load the cartridge and start the assay.