

Xpert® Xpress Flu/RSV Cartridge Preparation

- Xpert® Xpress Flu/RSV

for detailed instructions,
precautions, and warnings.

For a copy of the SDS, visit
www.cepheid.com or
www.cepheidinternational.com

Cepheid Technical Support

US office
(888) 838-3222, Option 2
techsupport@cepheid.com

European office
+33 563 825 319
support@cepheideurope.com



Refer to the package insert


- 1 Take one Xpert cartridge for each sample.



- 2 Invert the tube 5 times.




- 3 Open the cartridge lid.

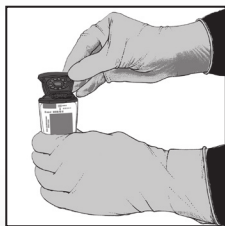

- 4 Using a clean 300 μ L pipette (supplied), transfer 300 μ L (one draw), of the sample to the opening of the cartridge.



- 5 Close the cartridge lid.

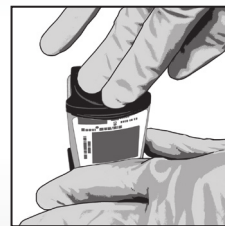

- 6 Start the test within the timeframe specified in the package insert.