

Xpert® Xpress Strep A Cartridge Preparation

- Xpert Xpress Strep A

Refer to the package insert for detailed instructions, precautions, and warnings.

For a copy of the SDS, visit www.cepheid.com or www.cepheidinternational.com

Cepheid Technical Support
US office
(888) 838-3222, Option 2
techsupport@cepheid.com

European office
+33 563 825 319
support@cepheideurope.com




1 Take one cartridge and a transfer pipette from the cartridge kit box.




2 Vigorously shake the transport tube for 5 seconds.



3 Open the cartridge lid.

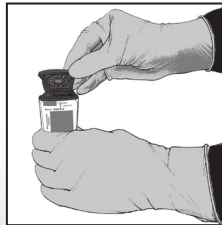

4 Open the cap on the specimen transport tube.

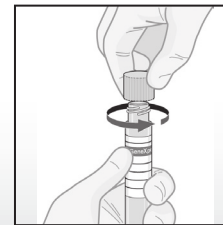

5 Using a clean 300µL pipette (supplied), transfer 300µL (one draw) of the sample to the opening of the cartridge.
NOTE: The pipette received may not be identical to the one shown.



6 Close the cartridge lid.

