

Xpert® Carba-R Cartridge Preparation using rectal or perirectal swab

Refer to the package insert for detailed instructions, precautions, and warnings.




For a copy of the SDS, visit www.cepheid.com or www.cepheidinternational.com

Cepheid Technical Support
US office
(888) 838-3222
techsupport@cepheid.com

European office
+33 563 82 53 19
support@cepheideurope.com



- 1 Obtain one Xpert cartridge and one Sample Reagent vial for each sample.
- 2 Insert the swab into the Sample Reagent vial.
- 3 Break the swab at the score mark at the score mark near the opening of the vial.
- 4 Recap the Sample Reagent vial and vortex for 10 seconds.
- 5 Open the Xpert cartridge lid.
- 6 Aspirate the Sample Reagent up to the line on the supplied pipette.
- 7 Empty the pipette into the sample chamber.
- 8 Close the Xpert cartridge lid.
- 9 Start the test within the timeframe specified in the package insert.

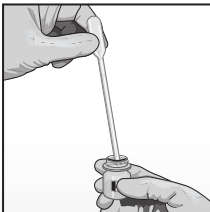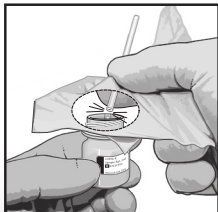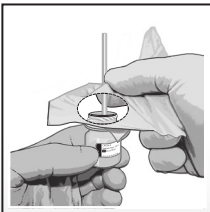

Note: Do not hold the swab below the score mark. Use gauze or its equivalent to minimize the risk of contamination.