

# Xpert<sup>®</sup> Cartridge Preparation

- Xpert Xpress SARS-CoV-2
- Xpert Xpress SARS-CoV-2/Flu/RSV
- Xpert Xpress CoV-2/Flu/RSV *plus*\*\*
- Xpert Xpress CoV-2 *plus*\*\*

Refer to the package insert for detailed instructions, precautions, and warnings.

For a copy of the SDS, visit [www.cepheid.com](http://www.cepheid.com) or [www.cepheidinternational.com](http://www.cepheidinternational.com)

Contact information for all Cepheid Technical Support offices is available on our website: [www.cepheid.com/en/CustomerSupport](http://www.cepheid.com/en/CustomerSupport).



\*\* Available for both CE-IVD & UKCA - IVD


- 1 Take one Xpert cartridge for each sample.



- 2 Rapidly invert the tube 5 times.




- 3 Open the cartridge lid.

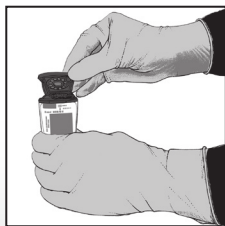

- 4 Using a clean 300  $\mu$ L pipette (supplied), transfer 300  $\mu$ L (one draw), of the sample to the opening of the cartridge.



- 5 Close the cartridge lid.

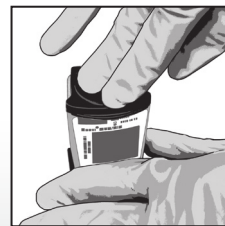

- 6 Start the test within the timeframe specified in the package insert.