

Xpert EV Cartridge Preparation

Refer to the package insert for detailed instructions, precautions, and warnings.

For a copy of the MSDS, visit www.cepheid.com or www.cepheidinternational.com

Cepheid Technical Support

US office
(888) 838-3222, Option 2
techsupport@cepheid.com

European office
+33 563 82 53 19
support@cepheideurope.com




- 1 Obtain one cartridge, three ampoules, and one vial for each sample.




- 2 Open the cartridge lid.



- 3 Open, and then add ampoule 1 into 1, ampoule 2 into 2, and ampoule 3 into 3.

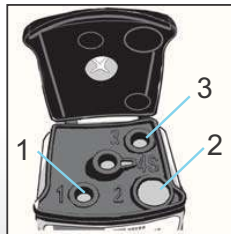

- 4 Pipette 140 μ L of the Lysis Reagent into chamber 4S, and then pipette 140 μ L of the sample into chamber 4S.

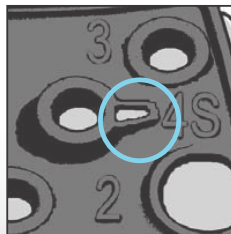

- 5 Close the cartridge lid.



- 6 Start the test within the timeframe specified in the package insert.