

Xpert® HPV Cartridge Preparation

Refer to the package insert for detailed instructions, precautions, and warnings.

For a copy of the SDS, visit www.cepheidinternational.com

Cepheid Technical Support

US office

(888) 838-3222

techsupport@cepheid.com


European office

+33 563 82 53 19

support@cepheideurope.com



1 Obtain one appropriately collected and labeled cervical sample.

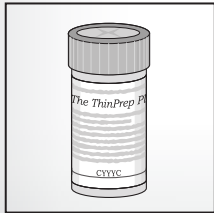

2 Obtain one Xpert HPV cartridge and transfer pipette (provided).




3 Open the Xpert cartridge lid.



4 Gently invert the sample vial 8-10 times, or vortex briefly with a vortex mixer at half-speed approximately for 5 seconds.



5 Fill the transfer pipette with sample up to the line (1ml).

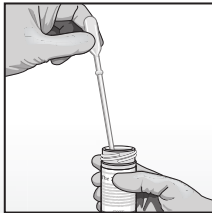

6 Empty the pipette contents into the sample chamber.



7 Close the Xpert cartridge lid.



8 Insert the cartridge and start the assay.

