

Xpert® HBV VL Cartridge Preparation

- Xpert® HBV Viral Load

Refer to the package insert for detailed instructions, precautions, and warnings.

For a copy of the SDS, visit www.cepheid.com or www.cepheidinternational.com

Cepheid Technical Support
US office
(888) 838-3222, Option 2
techsupport@cepheid.com

European office
+33 563 825 319
support@cepheideurope.com

Prior to beginning this test:


1. Centrifuge the sample to separate the plasma/serum and red blood cells per the manufacturer's instruction.
2. Equilibrate the plasma/serum to room temperature (20-35 °C).
3. Vortex for 10 seconds before use. If specimen is cloudy, clarify by a quick spin..




- 1 Take one Xpert cartridge for each sample.



- 2 Open the cartridge lid.

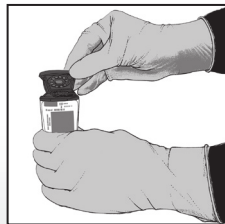

- 3 Aspirate the plasma/serum to just above the 1mL mark on the transfer pipette Or Aspirate just 0.6mL plasma/serum using precision pipette.

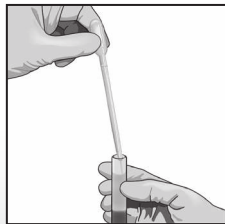

- 4 Empty the contents into the sample chamber.



- 5 Close the cartridge lid.



- 6 Start the test within 4 hours after adding the sample into the cartridge .