

# Xpert<sup>®</sup> Cartridge Preparation *depicting urine or swab sample in transport reagent tube.*

- Xpert CT/NG
- Xpert TV

**Refer to the package insert for detailed instructions, precautions, and warnings.**


For a copy of the SDS, visit  
[www.cepheid.com](http://www.cepheid.com) or  
[www.cepheidinternational.com](http://www.cepheidinternational.com)

Cepheid Technical Support  
US office  
(888) 838-3222, Option 2  
[techsupport@cepheid.com](mailto:techsupport@cepheid.com)

European office  
+33 563 82 53 19  
[support@cepheideurope.com](mailto:support@cepheideurope.com)



- 1 Obtain one appropriately collected and labeled transport reagent tube.

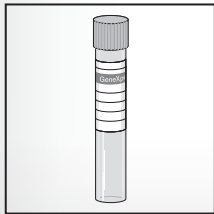


- 2 Obtain one Xpert cartridge and transfer pipette (provided in the assay kit).




- 3 Open the Xpert cartridge lid.



- 4 Gently invert the transport tube 3 to 4 times to mix.

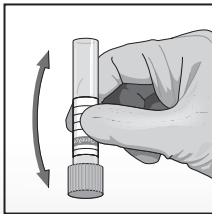


- 5 Fill the transfer pipette above the mark on the pipette shaft.

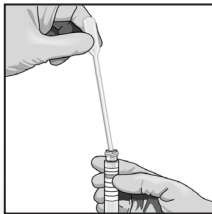

- 6 Empty the pipette contents into the sample chamber.



- 7 Close the Xpert cartridge lid.

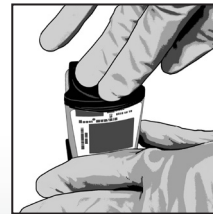

- 8 Insert the cartridge and start the test.



# Xpert<sup>®</sup> Cartridge Preparation *depicting urine sample in primary sample cup*

- Xpert CT/NG
- Xpert TV

**Refer to the package insert for detailed instructions, precautions, and warnings.**

For a copy of the SDS, visit [www.cepheid.com](http://www.cepheid.com) or [www.cepheidinternational.com](http://www.cepheidinternational.com)

Cepheid Technical Support  
US office  
(888) 838-3222, Option 2  
[techsupport@cepheid.com](mailto:techsupport@cepheid.com)


European office  
+33 563 82 53 19  
[support@cepheideurope.com](mailto:support@cepheideurope.com)




- 1 Obtain a first catch sample in primary collection cup. Ensure urine is well mixed in the urine cup before transferring aliquot to the Xpert Urine Transport Reagent tube.



- 2 Obtain one Xpert Urine Specimen Collection Kit.

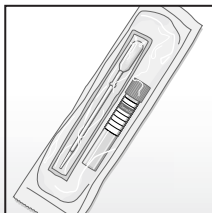

- 3 Open the packaging of a disposable transfer pipette provided in the Xpert Urine Specimen Collection Kit  
*NOTE: DO NOT use the pipette from the assay kit.*

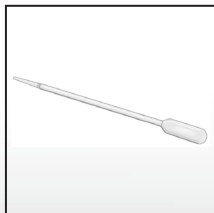


- 4 Insert the transfer pipette into the bottom of the urine cup.



- 5 Transfer approximately 7 mL of urine so that the volume reaches the black dashed line on the tube label.



- 6 Replace the cap on the Urine Transport Reagent tube cap and tighten securely. Label appropriately and transfer to the GeneXpert assay cartridge preparation area.

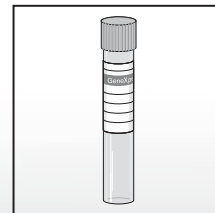

- 7 See reverse side of this card for further instructions.